


**BruxZir®**  
Solid Zirconia Crowns & Bridges

## Preparation Guide

# Use This Guide For BruxZir & Uni-Zirc

BruxZir is a 3Y full contour Zirconia indicated for crowns, bridges, veneers, inlays and onlays. It is a metal-free alternative to PFM or full-case restorations and ideal for cases requiring extra durability such as crowns under partials or screw-retained implant crowns. The chip-resistant durability of BruxZir makes it a good choice for bruxists who have broken their natural teeth or previous restorations through grinding and it can also work well for patients lacking the prep space for a PFM.

An alternative option with similar levels of strength but paired with enhanced aesthetics is Uni-Zirc. Uni-Zirc is a 4y full contour Zirconia featuring a built-in graduated translucency making it an excellent choice for anterior cases.

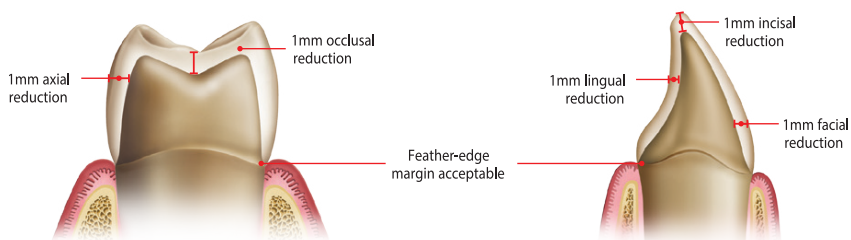

## Preparation Requirements

- Shoulder preparation not needed, feather edge is OK. Any preparation with at least 0.8 mm of occlusal space is accepted.
- Minimum occlusal space required is 0.8mm however more space means better crown shape and strength.

**CosTech® Dental Lab**

17-18 Lion Business Park, Dering Way, Gravesend, Kent DA12 2DN



[costech.co.uk](http://costech.co.uk)



[info@costech.co.uk](mailto:info@costech.co.uk)



01474 320076